



I10 TRAPEZE IMPLANT

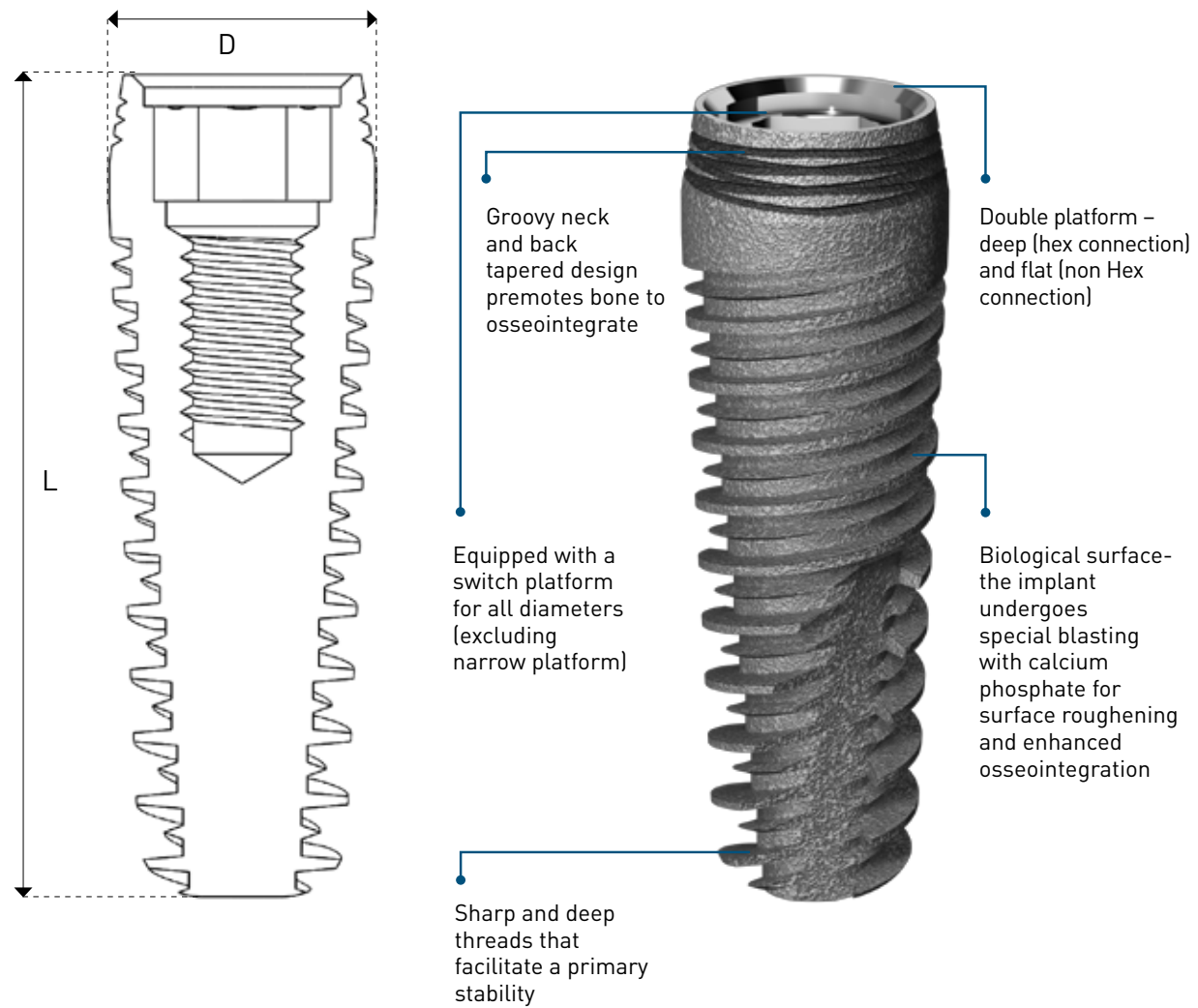

CAT no.	D (mm)	Platform	L (mm)	Tube top cap colors	With/Without Implant carrier
I10	3.75	Narrow	8, 10, 11.5, 13, 16	●	Both
I10	4.2	Standard	8, 10, 11.5, 13, 16	●	Both
I10	5	Standard	8, 10, 11.5, 13	○	Both

RECOMMENDED DRILL PROTOCOL FOR STRAIGHT OR STEP DRILLS

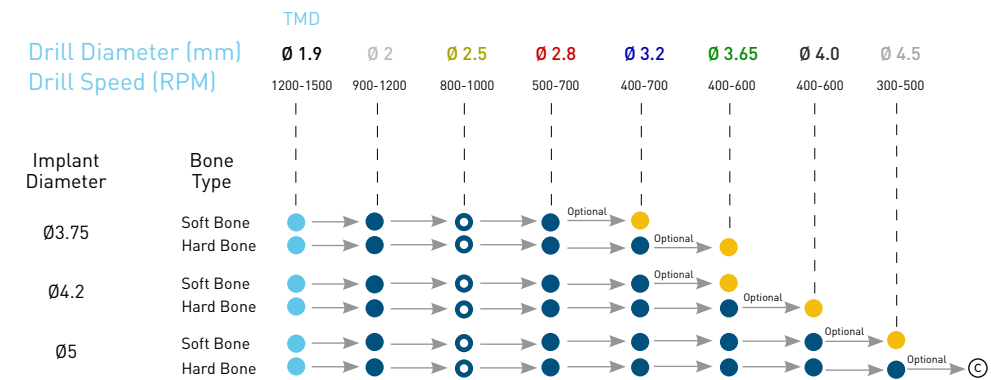


- Mark drill site
- Drill throughout entire implant's length
- Optional - In case TDSD step drill is used
- Drill through cortical plate in case needed
- Ⓢ Drill through cortical plate with Counter Sink drill in case needed

PACKAGE CONTENT AB Dental Implants can be provided in a package with or without an implant carrier.


WITH CARRIER



WITHOUT CARRIER

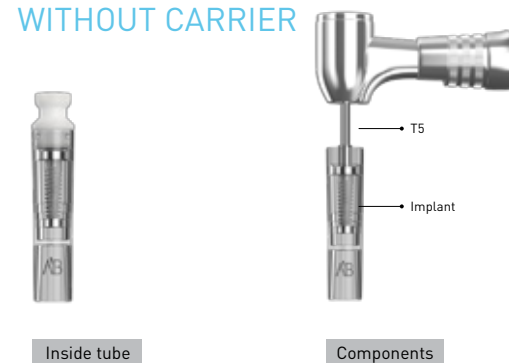

OPTIONAL DRILLS

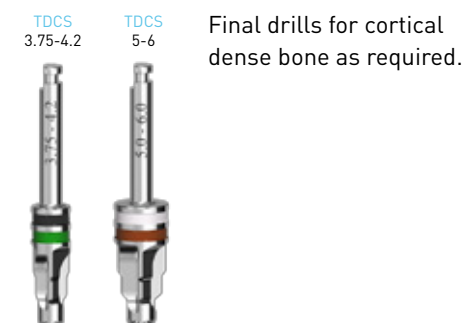

Procedure recommended by AB Dental should not replace the dentist/surgeon's judgment and experience. Final drill color (for hard bone) should correspond to Implant's Tube Cap color.